

HAMILTON-C6

Technical specifications for SW version 1.1.x

Ventilation modes

Mode form	Mode name	Mode	Adult/Ped	Neonatal
Volume-controlled, flow-controlled	(S)CMV	Breaths are volume controlled and mandatory, including patient triggered breaths.	✓	--
	SIMV	A fixed rate is set for volume-controlled mandatory breaths. Additional patient triggered breaths between mandatory breaths are spontaneous breaths (with or without pressure support).	✓	--
Volume-targeted, adaptive pressure- controlled	APVcmv / (S)CMV+	Breaths are volume targeted and mandatory.	✓	✓
	APVsimv / SIMV+	Volume-targeted mandatory breaths can be alternated with pressure-supported spontaneous breaths.	✓	✓
Pressure-controlled	PCV+	All breaths, whether triggered by either the patient or the ventilator, are pressure controlled and mandatory.	✓	✓
	P-SIMV+	Mandatory breaths are pressure controlled. Mandatory breaths can be alternated with pressure-supported spontaneous breaths.	✓	✓
	DuoPAP	Mandatory breaths are pressure controlled. Spontaneous breaths can be triggered at both pressure levels.	✓	✓
	APRV	Spontaneous breaths can be continuously triggered. The pressure release between the levels contributes to ventilation.	✓	✓
	SPONT	Every breath is spontaneous, with or without pressure support.	✓	✓
Intelligent ventilation	ASV®	Operator sets %MinVol, PEEP, and Oxygen. Frequency, tidal volume, pressure, and I:E ratio are based on physiological input from the patient.	✓	--
	INTELLiVENT®-ASV	Fully automated management of ventilation and oxygenation based on physiological input from the patient. The underlying mode is ASV.	0	--
Noninvasive ventilation	NIV	Every breath is spontaneous, with or without pressure support.	✓	✓
	NIV-ST	Every breath is spontaneous as long as the patient is breathing above the set rate. A backup rate can be set for mandatory breaths.	✓	✓
	nCPAP-PS	Every breath is spontaneous as long as the patient is breathing above the set rate. A backup rate can be set for mandatory breaths. PS can be set to 0 (= nCPAP)	--	0
Oxygen therapy	HiFlowO2	High flow oxygen therapy. No supported breaths.	0	0

Standard: ✓ Option: 0 Not applicable: --



Standard configuration and options (in alphabetical order)

Functions	Adult / Ped	Neonatal
Capnography, mainstream (volumetric) and sidestream	O	O
Communication ports: Three COM ports, two USB ports, DVI, Nurse call	✓	✓
Communication protocols: for details see Connectivity brochure	✓	✓
Dynamic Lung (real-time visualization of the lungs)	✓	--
Event log (up to 10,000 events with date and time stamp)	✓	✓
HAMILTON-H900 humidifier control via ventilator	O	O
Inspiratory and expiratory hold maneuver	✓	✓
IntelliCuff® cuff pressure controller control via ventilator	O	O
IntelliSync+ (inspiratory and expiratory trigger synchronization)	O	--
IntelliTrig (leak compensation)	✓	✓
Manual breath / prolonged inspiration	✓	✓
Nebulization (Aerogen [®])	O	O
Nebulization (pneumatic)	✓	--
O2 enrichment	✓	✓
On-screen help	✓	✓
P/V Tool® Pro	O	O
Paramagnetic O2 sensor	O	O
Patient group	✓	O
Print screen	✓	✓
Screen lock	✓	✓
Second battery	O	O
SpO2 monitoring	O	O
Standby with timer	✓	✓
Suctioning tool	✓	✓
Transpulmonary pressure monitoring	✓	✓
TRC (tube resistance compensation)	✓	✓
Trends/Loops	✓	✓
Trigger, flow and pressure selectable	✓	✓
Vent Status (Visual representation of ventilator dependency)	✓	✓

Standard: ✓ Option: O Not applicable: --

Technical performance data (in alphabetical order)

Description	Specification
Automatic expiratory base flow	Fixed at 6 l/min
Inspiratory pressure	0 to 100 cmH ₂ O
Maximum inspiratory flow	260 l/min
Means of inspiratory triggering	Flow trigger control, pressure trigger control, or optional IntelliSync+ control
Means of expiratory triggering	Flow cycle (ETS), or optional IntelliSync+ control
Minimum expiratory time	20% of cycle time; 0.2 to 0.8 s
O ₂ input flow	150 l/min (at 2.8 bar/ 280 kPa / 41 psi input pressure)
Oxygen mixer accuracy	± (Volume fraction of 2.5% + 2.5% of actual reading)
Preoperational checks	Tightness test, flow sensor/O ₂ sensor/CO ₂ sensor calibration
Tidal volume	Adult/Ped: 20 to 2000 ml Neonatal: 2 to 300 ml

Standards and approvals

Classification	Class IIb, continuously operating according to EC directive 93/42/EEC
Certification	EN 60601-1:2006/A1:2013, IEC 60601-1-2:2014, ANSI/AAMI ES60601-1:2005/(R)2012, ISO 80601-2-12:2011, CAN/CSA-C22.2 NO. 60601-1:14, EN ISO 5356-1:2015, ISO 80601-2-55:2011
Declaration	The HAMILTON-C6 was developed in accordance with pertinent international standards and FDA guidelines. The ventilator is manufactured within an EN ISO 13485 and EN ISO 9001, Council Directive 93/42/EEC, Annex II, Article 3 certified quality management system. The ventilator meets the Essential Requirements of Council Directive 93/42/EEC, Annex I.
Electromagnetic compatibility	According to IEC 60601-1-2:2014
Safety Class	Class I, Type B applied part (ventilator breathing system, VBS), type BF applied parts CO ₂ sensor including CO ₂ module connector, humidifier, Aerogen ^s system, nebulizer, and SpO ₂ sensor including SpO ₂ adapter, continuous operation according to IEC 60601-1

Pneumatic specifications

O2	Input pressure	2.8 to 6 bar / 41 to 87 psi
	Connector	DISS (CGA 1240) or NIST
Air supply		Integrated turbine with lifetime warranty
Inspiratory outlet (To patient port)	Connector	ISO 15 mm ID/22 mm OD conical
Expiratory outlet (From patient port)	Connector (on expiratory valve)	ISO 15 mm ID/22 mm OD conical

Electrical specifications

Input power	100 to 240 VAC ±10%, 50/60 Hz	
Power consumption	60 W typical, 210 W maximum	
Battery	Electrical specifications:	14.4 V, 6.8 Ah, 98 Wh, 35 W typical, 115 W maximum
	Type:	Lithium-ion
	Normal operating time:	≥ 90 min with one battery / ≥ 180 min with two batteries

Graphical patient data

Graphic type/Tab name	Options
Waveforms	Pressure, Flow, Volume, Off, PCO ₂ ¹ , FCO ₂ ¹ , Plethysmogram ¹ , Ptrachea, Pes, Ptranspulm
Intelligent panels	Dynamic Lung ² , Vent Status, ASV Graph ³ , SMPs (Secondary monitoring parameter)
Trends	1-, 6-, 12-, 24-, or 72-h trend data for a selected parameter or combination of parameters
Loops	Pressure/Volume, Pressure/Flow, Volume/Flow, Volume/PCO ₂ ¹ , Volume/FCO ₂ ¹ , Pes/Volume, Ptranspulm/Volume

Alarms⁴

Priority	Alarm
High priority	Apnea time (s), ExpMinVol high/low (l/min), Oxygen high/low (%), Pressure high/low (cmH ₂ O), Flow sensor calibration needed, Exhalation obstructed, Disconnection, Oxygen supply failed
Medium priority	fTotal high/low (b/min), PetCO ₂ high/low (mmHg), Pressure limitation (cmH ₂ O), Vt high/low (ml), SpO ₂ high/low, SpOC high/low, %leak, High PEEP, Loss of PEEP, Pulse high/low
Low priority	High SpO ₂ , Loss of external power, Cuff leakage

¹ CO₂ + SpO₂ option required | ² For adult/pediatric patients only | ³ Only available in ASV mode | ⁴ For complete list of alarms see operation manual

Control settings and ranges⁵

Parameter (units)	Range Adult/Ped	Range Neonatal
Apnea backup	On, Off	On, Off
Cuff pressure (cmH2O)	0 to 50	0 to 50
Expiratory trigger sensitivity ETS (%)	5 to 80	5 to 80
Flow for HiFlowO2 therapy (l/min)	2 to 80	2 to 12
Flow pattern	Square, 50% decelerating, Sine, 100% decelerating	--
Flow trigger (l/min)	0.5 to 20, off	0.1 to 5.0, off
Gender	Male, Female	--
I:E	1:9 to 4:1	1:9 to 4:1
%MinVol (%)	25 to 350	--
Oxygen (%)	21 to 100	21 to 100
P high (cmH2O) (only in DuoPAP and APRV)	0 to 100	0 to 60
P low (cmH2O) (only in APRV)	0 to 50	0 to 25
Pasvlimit (cmH2O)	5 to 100	5 to 100
Pat. height (cm) (in)	30 to 250 / 12 to 98	--
Pause (%)	0 to 70	--
Pcontrol (cmH2O)	5 to 100	3 to 60
Peak flow (l/min)	1 to 195	--
PEEP/CPAP (cmH2O)	0 to 50	0 to 25
Pinsp (cmH2O)	3 to 100	0 to 60
P-ramp (ms)	0 to 2000	0 to 600
Pressure trigger (cmH2O)	-0.1 to -15.0, off	-0.1 to -15.0, off
Psupport (cmH2O)	0 to 100	0 to 60
Rate (b/min)	1 to 80	1 to 150
Sigh	On, Off	--
T high (s) (only in DuoPAP und APRV)	0.1 to 40	0.1 to 40
T low (s) (only in APRV)	0.2 to 40	0.2 to 40
TI (s)	0.1 to 12	0.1 to 12
TI max (s)	1 to 3	0.25 to 3.0
Tip (s)	0 to 8	--
Tpause (s)	0 to 30	0 to 30
TRC compensation (%)	0 to 100	0 to 100
Vt (ml)	20 to 2000	2 to 300
Weight (kg)	--	0.2 to 30.0

⁵ Parameter settings and ranges can change depending on the mode

Monitoring parameter

Parameter (units)		Description	
Pressure	AutoPEEP (cmH ₂ O)	Unintended positive end-expiratory pressure	
	Paw (cmH ₂ O)	Airway pressure	
	ΔP (cmH ₂ O)	Driving pressure	
	P0.1 (cmH ₂ O)	Airway occlusion pressure	
	PTP (cmH ₂ O*s)	Inspiratory pressure time product	
	Pcuff (cmH ₂ O)	Cuff pressure	
	Ptrans I (cmH ₂ O)	The arithmetic mean value of Ptranspulm over the last 100 ms of the last inspiration.	
	Ptrans E (cmH ₂ O)	The arithmetic mean value of Ptranspulm over the last 100 ms of the last expiration.	
	PEEP/CPAP (cmH ₂ O)	PEEP (positive end-expiratory pressure) and CPAP (continuous positive airway pressure)	
	Pinsp (cmH ₂ O)	Inspiratory pressure	
	Pmean (cmH ₂ O)	Mean airway pressure	
	Ppeak (cmH ₂ O)	Peak airway pressure	
	Pplateau (cmH ₂ O)	Plateau or end-inspiratory pressure	
	Pes min (cmH ₂ O)	See PEEP. The pressure is measured through the Pes port instead of using airway pressure.	
	Pes max (cmH ₂ O)	See Ppeak. The pressure is measured through the Pes port instead of using airway pressure.	
	Flow	Pes plateau (cmH ₂ O)	See Pplateau. The pressure is measured through the Pes port instead of using airway pressure.
Pes PTP (cmH ₂ O)		See PTP. The pressure is measured through the Pes port instead of using airway pressure.	
Pes P0.1 (cmH ₂ O)		See P0.1. The pressure is measured through the Pes port instead of using airway pressure.	
Control Flow (l/min)		The set flow of gas to the patient. HiFlowO ₂ mode only.	
Insp Flow (l/min)		Peak inspiratory flow, spontaneous or mandatory	
Exp Flow (l/min)		Peak expiratory flow	
Volume		ExpMinVol or MinVol NIV (l/min)	Expiratory minute volume
		MVSpont or MVSpont NIV (l/min)	Spontaneous expiratory minute volume
	VTE or VTE NIV (ml)	Expiratory tidal volume	
	VTESpont (ml)	Spontaneous expiratory tidal volume	
	VTI or VTI NIV (ml)	Inspiratory tidal volume	
	Vt/IBW	Tidal volume according to ideal body weight (IBW) for adult/ pediatric patients and	
	Vt/Weight (ml/kg)	according to the actual body weight for neonatal patients.	
	VLeak (%) or MVLeak (l/min)	Leakage percent or total minute volume leakage	

Monitoring parameter (continued)

Parameter (units)	Description		
CO2	FetCO2 (%)	Fractional end-tidal CO2 concentration	
	PetCO2 (mmHg)	End-tidal CO2 pressure	
	slopeCO2 (%CO2 / l)	Slope of the alveolar plateau in the PetCO2 curve, indicating the volume/flow status of the lungs	
	Vtalv (ml)	Alveolar tidal ventilation	
	V'alv (l/min)	Alveolar minute ventilation	
	V'CO2 (ml/min)	CO2 elimination	
	VDaw (ml)	Airway dead space	
	VDaw/VTE (%)	Airway dead space fraction at the airway opening	
	VeCO2 (ml)	Exhaled CO2 volume	
	ViCO2 (ml)	Inspired CO2 volume	
	SpO2	SpO2 (%)	Oxygen saturation
		Pulse (1/min)	Pulse
		Plethysmogram	The waveform that visualizes the pulsating blood volume; it is delivered by the pulse oximeter.
SpO2/FiO2 (%)		The SpO2/FiO2 ratio (%) is an approximation of the PaO2/FiO2 ratio, which, in contrast to PaO2/FiO2, can be calculated noninvasively and continuously.	
PI (%)		Perfusion index	
PVI (%)		Pleth variability index	
SpCO (ml/dl) ² (%) ²		Carboxyhaemoglobin saturation	
SpMet (%)		Methaemoglobin saturation	
SpHb (g/dl) (mmol/l)		Total haemoglobin	
SpOC (ml/dl)		Oxygen content	
Oxygen		Oxygen (%)	Oxygen concentration of the delivered gas
Time	I:E	Inspiratory:expiratory ratio	
	fControl (b/min)	Mandatory breath frequency	
	fSpont (b/min)	Spontaneous breathing frequency	
	fTotal (b/min)	Total breathing frequency	
	TI (s)	Inspiratory time	
	TE (s)	Expiratory time	
	Pause (s)	Inspiratory pause or plateau	
Lung mechanics	Cstat (ml/cmH2O)	Static compliance	
	P0.1 (cmH2O)	Airway occlusion pressure	
	PTP (cmH2O*s)	Pressure time product	
	RCexp (s)	Expiratory time constant	
	Rinsp (cmH2O/(l/s))	Inspiratory flow resistance	
	RSB (1/(l*min))	Rapid shallow breathing	



Physical characteristics

Weight	<p>Monitor (interaction panel) 7.8 kg (17.2 lb), with shelf mount: 10.0 kg (22.0 lb)</p> <p>Ventilation unit, shelf mount: 10.5 kg (23.15 lb)</p> <p>46 kg (101 lb) with trolley, monitor, ventilation unit</p> <p>The trolley can accommodate a maximum safe working load of 80 kg (176 lb)</p>
Dimensions	See graphic above
Monitor	Type: Color TFT, Size: 1920 x 1200 pixels, 17 in (431.8 mm) diagonal
Monitor mounting options	VESA, pole mount, rail mount, handle mount
Trolley accessories	Basket for suctioning catheter, O2 bottle holding system, HAMILTON-H900 mounting system

Manufacturer:

Hamilton Medical AG

Via Crusch 8, 7402 Bonaduz, Switzerland

+41 (0)58 610 10 20

info@hamilton-medical.com

www.hamilton-medical.com

689596.01

Specifications are subject to change without notice. Some features are options. Not all features are available in all markets. INTELLIVENT-ASV is not available in the US. For all proprietary trademarks (®) and third-party trademarks (®) used by Hamilton Medical AG see www.hamilton-medical.com/trademarks. © 2017 Hamilton Medical AG. All rights reserved.

HAMILTON-C6