

evac-s TIMESCO

A manual, hand held
single use suction device.



The EVAC-S is a completely single-use, manual hand held Suction Device, which is supplied with 300ml Canister, Adult & Paediatric Catheters and a self-seal disposal bag and Bio-Hazard label. It is lightweight, portable and offers a high vacuum performance (500 mmHg), The EVAC-S can be powered with one hand operation, leaving the other hand free for other tasks.

- **Completely Single-Use**

No risk of cross contamination, infection control friendly.

- **Easy Disposal**

The EVAC-S is supplied in a self-seal polybag with two ply label. After use place the device into the self-seal polybag, re-seal and peel away the product label to reveal a Bio-Hazard warning label. Dispose of in the correct manner, as per your agreed protocols.

- **Adult & Paediatric Use**

The EVAC-S is supplied with two catheters, an Adult 9mm (I/D) catheter and a 10fg Paediatric catheter.

- **100 Degree Swivel Action**

The EVAC-S handle was designed with a 100 degree swivel action to allow the user better accessibility whilst working in difficult situations.

- **Overflow Mechanism & One Way Valves**

The EVAC-S is equipped with one way valves either side of the Canister cap and an overflow protection mechanism which will engage once the canister is full. It's not possible to continue squeezing at this point (strong resistance will be felt), both will prevent any leakage or spillage from the canister.

- **High Power**

When used with the Adult Catheter The EVAC-S can generate a maximum vacuum of up to 500mmHg

- **Specification:**

- Vacuum (max): -500 mmHg
- Peak free airflow: >20ml
- Disposable container volume: 300 ml
- External diameter of cap (suction port): 17 mm
- External diameter of catheters: 9mm Adult, 10fg Paediatric
- Operating environmental temperature: -20°C to +50°C
- Storage environmental temperature: -40°C to +60°C
- Net weight: Nett 270 grams

Product Code: TDM-ESD-100

Description: EVAC-S Single-Use Suction Device (Pack of 5)



After use place the device into the self-seal polybag, re-seal and peel away the product label to reveal a Bio-Hazard warning label.



Supplied as boxes of 5. Each EVAC-S contains instructions for use, 300ml Canister, Adult & Paediatric Catheters and a self-seal disposal bag which has a Bio-Hazard label.

Medidyne